

# TRI-SOLFEN<sup>®</sup> 2mL Applicator

## INSTRUCTIONS FOR USE

### PRIMING THE APPLICATOR

Set the applicator to the maximum dose, hold vertically and squeeze to draw the product into the cylinder (holding vertical will ensure that all the air is expelled). When a small amount of product appears at the spray tip, the applicator is ready for use. Ensure that the applicator is pointed away from yourself and others.

### TO ADJUST THE DOSE

Loosen the lock nut, screw the dose adjustor to align the front of the o-ring with the desired dose marking on the cylinder and retighten the lock nut.

### CLEANING & LUBRICATING

After each use, flush the applicator and connecting tube thoroughly by pumping through a warm water detergent mix, followed by clean water. Remove the feed tube and suck a small quantity of light oil, such as castor oil or vegetable oil, into the cylinder by immersing the inlet fitting in the oil. Gently squeeze the handle and push grip to suck oil into and through the applicator.

### STERILIZATION

After cleaning and lubricating the applicator, fill a deep pot with water. Attach connecting tube to applicator and pump handle to draw water into the cylinder. Suspend the whole applicator into the water but make sure it does not touch the bottom of the pot, add needles and boil for 10 - 20 minutes. Protect your hands by using a cloth to remove the applicator from the pot. Pump water out of cylinder and pat applicator dry. **Take care when you remove the applicator as contents will be hot.**

**Note: It is neither necessary, nor desirable, to dismantle the handle or any other component, before sterilizing.**

 **prodigy  
instruments**

MANUFACTURED BY  
prodigyinstruments.com.au

Thank you for your purchase of this product, as it continues a family tradition since 1931. From my grandfather Norman Phillips, to my father John Phillips and now to me, Matthew Phillips...the tradition lives on.

FOR TECHNICAL SERVICE, CONTACT MATTHEW PHILLIPS  
M: +61 403 454 354

PI25-BAYER-TRI | PL20-R2

# TRI-SOLFEN<sup>®</sup> 2mL Applicator

## INSTRUCTIONS FOR USE

### PREPARING THE APPLICATOR


Attach the connecting tube to both the hand piece and the draw-off system.

### TO CHANGE THE NOZZLE


- ① Unscrew the nozzle nut to release the nozzle.



- ② Place a new nozzle into inner groove of the nozzle nut.

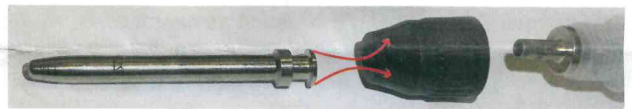

- ③ Tighten the nozzle nut securely in place.

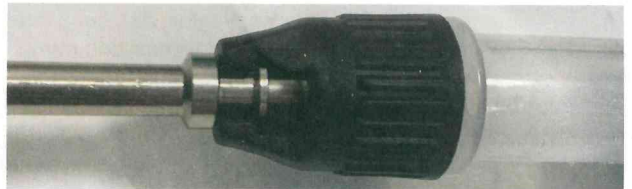

### STANDARD FAN-SPRAY NOZZLE

Orientation of nozzle changes direction of fan spray.



### CALF CASTRATION FAN-SPRAY NOZZLE

This symbol shows the direction of the fan spray.

